


User Guide

USER CONTROLS AND SYMBOLS

Coil connector socket allows a treatment coil to be safely and securely locked into place.

Preset 5 min. and 10 min. pushbuttons: press to begin a treatment. Blue lights illuminate when a cycle begins and turn off when it ends.

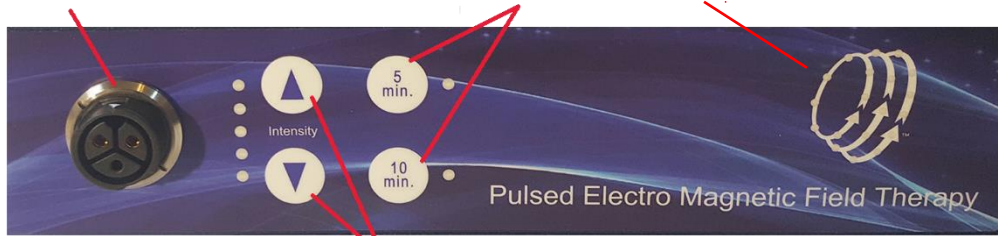

MAINS socket to plug in the medical grade power cord. The MAINS power rocker switch turns the device on "I" and off "O".

Intensity pushbutton up and down arrows; press ▲ to increase or ▼ to decrease the intensity to one of the five preset intensity levels. One to five blue lights will illuminate.

Treatment Cycle

A treatment cycle lasts 5 min. or 10 min. Press the desired pushbutton to begin a treatment. Each treatment cycle may be set at one of five intensity levels by pressing the up and down arrows. A corresponding number of blue lights will illuminate.

Getting Started

- ⚠ Connect the power cord into the MAINS socket on the back of the device.
- ⚠ Always lock a treatment coil connector into the connector socket on the front of the device before turning the MAINS power switch on "I". The device will not start if the treatment coil is not properly connected.

Inserting/Changing a Treatment Coil

To connect a treatment coil, align the arrow on the coil connector with the arrow on the coil connector socket of the device. Push the connector into the socket until you hear a "click". The treatment coil is locked into place. To remove the treatment coil, turn the end of the coil connector to the left (counter-clockwise) to disengage and pull it out. Do not plug or unplug the treatment coil during an active treatment cycle.

Beginning Treatment

Press the MAINS power rocker switch to "I" to turn the device on.

Place a treatment coil on or around the desired treatment area. The closer the coil is to the treatment area, the more effective the treatment cycle will be. Users can remain fully clothed, no direct contact between the coil and the skin is necessary for the treatment to be effective.

Press the 5 min. or 10 min. pushbutton to begin the treatment. Blue lights spiral when a treatment cycle begins. The blue lights turn off and the device emits a single long beep when the treatment cycle ends. Press a 5 min. or 10 min. pushbutton again to begin another treatment cycle.

Pausing/Resetting/Ending the Treatment

During the treatment cycle, briefly press the active treatment pushbutton to pause the treatment. The corresponding blue light flashes continuously while in "paused mode". To resume the treatment cycle, briefly press the same pushbutton again.

To end the treatment cycle prior to completion, press and hold the active treatment pushbutton until you hear a single long beep and the blue lights turn off.

FREQUENCY OF USE

The treatment cycle lasts 5 min. or 10 min. with each treatment coil.

On the first day, the recommended treatment is one 5 min. cycle per area treated. Thereafter, the treatment can be increased to one 10 min. treatment per area treated per day.

Additional treatments of the same area on the same day are not hazardous but do not offer additional therapeutic benefits.

ADVERSE REACTIONS

There are no known negative side effects, or reported adverse or allergic reactions with the use of this device.

Detoxification may occur, drink plenty of water after treatment.

Overuse may cause muscle soreness.

In case of any adverse reaction, stop using the device and consult a physician.

TROUBLESHOOTING

If the device fails to function, use the following troubleshooting steps:

If there are beeping sounds:

Proceed to identify the error beep code the device emits.

Single two second beep:

- End of session.

Solution: Press a start pushbutton to begin a new treatment.

Three short beeps & one long beep:

- The coil is not connected properly or was disconnected while the device was in use.

Solution: Connect the coil properly and press a start pushbutton to begin a treatment.

Two short beeps three times, followed by one long beep:

- Device overheating.

Solution: Allow the device to cool down for 10 minutes by leaving the power on while no treatment is activated. The cooling fan will cool the device. Press a start pushbutton to begin a treatment after the 10 minutes have elapsed.

If the lights do not illuminate and there are no beeping sounds:

- Make sure the MAINS power cord is properly connected at the control unit and a MAINS outlet.
- Make sure the MAINS outlet is functioning. Try plugging another electrical appliance into the outlet to determine if it works. If the outlet works, test the device with another MAINS power cord.

If the device still fails to start or function properly, contact the manufacturer, listed on the enclosed Warranty Card to get a Return Merchandise Authorization (RMA) number. Disconnect the coil and the power cord. Package the device securely and return it to the manufacturer for servicing with the RMA number and your contact information.

STORAGE AND TRANSPORTATION

Keep the device dry and store in a dry place. Storage in a damp place may cause corrosion.

Store and transport at temperatures between 10° to 50°C at a relative humidity of up to 93% non-condensing.

Remove the MAINS power cord and the detachable treatment coils from the control unit and pack all parts of the device securely before transporting.

DEVICE SPECIFICATIONS

Voltage input:	120 to 230 VAC, 50 to 60Hz
Current input:	1 Amp maximum
Dimensions:	10.83" x 6.89" x 2.56"
Weight:	5.8 lbs
Shipping weight:	13.5 lbs